



PARAMOUNT BED CO., LTD.
2-14-5 Higashisuna Koto-Ku, Tokyo 136-8670, Japan
Tel: +81 3 3648 2961 Fax: +81 3 3648 2420
<http://www.paramount.co.jp/english/>




Celenus™ Series
Delivery Bed

celebrating
the start of a
beautiful life journey

Designs and specifications are subject to change without notice. All images of the product, its components and accessories may not reflect the final product. The images may reflect optional equipment and are subject to change without notice. All information presented herein is based on current data or design goals available at the time of posting and is subject to change without notice. Some equipment and/or features may be unavailable. Actual results of performance and other specifications may differ or vary with production models and may depend on selected models and/or accessories.

A151-1601-EN01

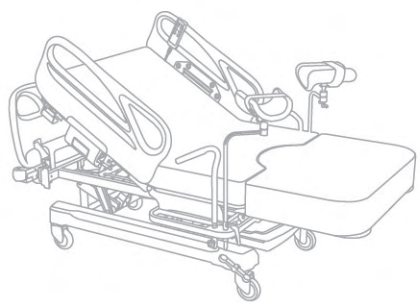

Celebrating a Beautiful Journey

Nothing brings more joy to a family than the birth of a child. The new parents are filled with anticipation and excitement at the thought of welcoming a new family member to the world. That first cry of life and that little smile will not only capture the hearts of Mom and Dad, but also the hearts of medical personnel, creating the most beautiful and unforgettable memories. Paramount Bed is privileged to share in this wonderful celebration of a new life journey.

Naturally, the roles of both the mother and medical personnel are critical for a smooth delivery. That is why Paramount Bed has designed Celenus™ to meet their needs.

Designed to ensure that the safety needs of both mother and medical personnel are met, Paramount Bed's Celenus™ also:

- Offers ease and comfort for the mother during the birth process.
- Fulfills maternity care functions and enables medical personnel to perform their duties in a fast and efficient manner.

The delivery bed that understands your needs

Delivering Care

The mother's comfort and the medical personnel's work efficiency are the primary concerns for a smooth delivery process. With Paramount Bed's Celenus™, we present a variety of features catered to these needs.

For Mothers

Delivery Positions

The mother may opt for her preferred delivery position.



Celenus™ Features

Features of Celenus™ that can aid the mother in the delivery process.

01

Anchor Support

After back raise, the Trendelenburg function can also be activated to create an anchor support to prevent slippage towards the foot of the bed.



02

Calf Supports / Foot Rests*

The calf supports are easily adjustable and cushioned, minimising stress on the mother's legs. The mother may use the foot rests for birth support other than the calf supports.



* Available as an additional accessory

03

Side-rail

There are many positions on the side-rail where the mother can hold on to for support. The side-rail moves along with the back section of the bed during back raise, allowing the mother to maintain her grip on it without having to re-adjust.



04

Wide Bed Width

At a width of 910mm, Celenus™ provides the mother with ample space to care for her baby in the bed.



Facilitating Performance

Durable, lightweight and excellent in design, the user-friendly and ergonomic Celenus™ boasts an assortment of functions to allow medical personnel to carry out their work with ease.

For Medical Personnel

01

Leg Section Retraction Lever

The leg section retraction lever allows the leg section base of the bed to be stowed easily under the bed without the need to be entirely removed. This speeds up preparation for delivery and improves efficiency.



Leg section base before stowing



Pull up lever to stow



Leg section base fully retracted

02

Foot Support Bar*

Medical personnel can place their feet on the foot support bar for better leverage support while delivering the baby.



* Available as an additional accessory

03

Fluid Basin

A high capacity 16L fluid basin facilitates the easy collection of birth fluids. It features good side grips so it can be carried with ease. Vinyl bags can also be placed over it for fluid disposal. The fluid basin can be easily removed for washing.



04

Manual CPR Lever

The manual CPR lever can be used to quickly return the raised back section to the horizontal position during an emergency or power failure.



Model No. PD-200 B/D/E*

Total Length	2,072mm
Total Width**	1,056mm
Total Height	883 to 1,213mm
Mattress Base Length	1,910mm (normal bed length)
Mattress Base Width	910mm
Mattress Base Height	500 to 830mm
Dimensions of Head Mattress	W910mm x L1,330mm x T130mm
Dimensions of Foot Mattress	W910mm x L705mm x T175mm
Trendelenburg	0° to 12°
Back Raise Angle	0° to 65°
Safe Working Load***	220kg
Battery	Yes
Caster	125mm dia. single-wheel casters with total locking system

CE * B: Blue / D: Green / E: Pink
 ** When side-rails are raised.
 *** Maximum load with which the bed can be operated.

Note: Back raise / Hi-lo / Trendelenburg functions are motorized.



Model No. PD-100 B/D/E*

Total Length	2,010mm
Total Width	960mm
Total Height	1,088mm
Mattress Base Length**	1,910mm
Mattress Base Width	830mm
Mattress Base Height	690mm
Dimensions of Head Mattress	W830mm x L1,330mm x T80mm
Dimensions of Foot Mattress	W830mm x L705mm x T80mm
Back Raise Angle	0° to 80°
Safe Working Load***	204kg
Caster	75mm dia. double-wheel casters with stopper

CE * B: Blue / D: Green / E: Pink
 ** When the foot frame is attached.
 *** Maximum load with which the bed can be operated.



Other Features



Handheld Controller (Available on PD-200)

Convenient placement on the side-rail and main frame for easy access by mother and medical personnel.

- Back raise / Hi-lo / Trendelenburg can be performed using the handheld controller
- During delivery, the handheld controller can be hung on the main frame for easy access by the medical personnel at times when they are at the center or foot of the bed
- After delivery, while the mother is recovering, the handheld controller can be hung on the side-rail so that the mother can self-adjust the bed to make herself comfortable

Additional Accessories

PD-200

- Foot Rests (short / long)
- Foot Support Bar
- Oxygen Cylinder Holder (can hold up to 1000L)
- IV Pole

PD-100

- Hand Grips
- Accessory Storage Tray
- Oxygen Cylinder Holder (can hold up to 1000L)
- IV Pole

Headboard

- Removable Headboard
- Auto-lock System



PD-200

PD-100

Color Options

Standard Color - Pink
Made to Order - Blue and Green



Mattress

Surface Material: Vinyl Leather

- MRSA anti-bacterial treated


Core Material: Urethane Foam

Double-stitching

- Prevents fluid seepage more effectively as compared to single-stitching process

Flame Retardant

Water Proofed

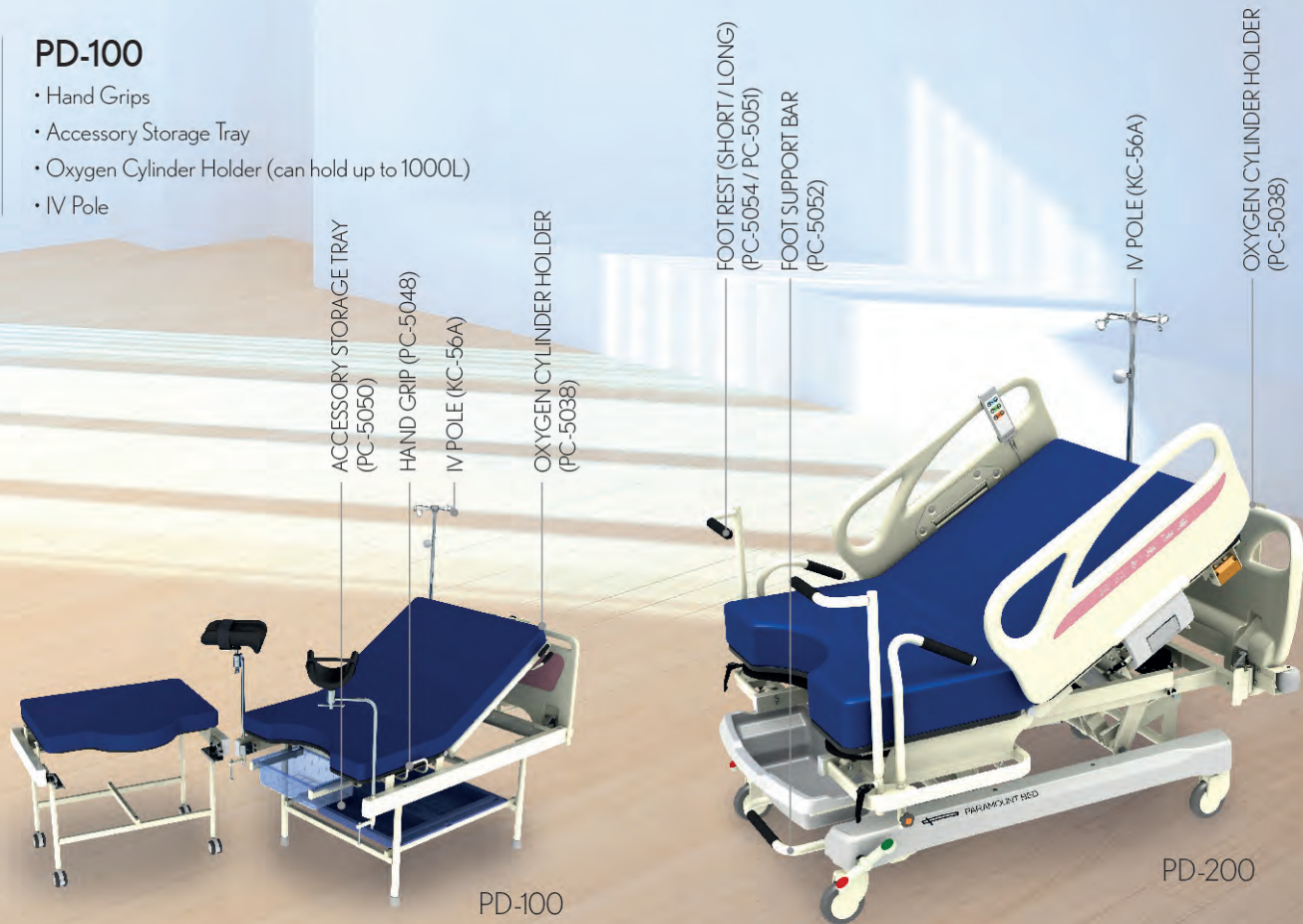

Bassinet Cart

When the little one completes the journey into the world, there comes a time to rest. Paramount Bed's bassinet carts are designed to ensure that the baby's sweet beginnings are secure and comfortable, while enabling medical personnel to enjoy the efficiency and reliability of an ergonomically designed bassinet to facilitate their care of the newborn.



KB-115E / 116E
CE

The KB-115E / 116E bassinet carts are the ideal tools of recuperation for the precious one, with their organic design taking inspiration from the leaves of nature. The baby is nestled in a bassinet that is fitted with thoughtful functions such as a wider opening with rounded corners, allowing caregivers to easily approach and handle the newborn.

For the KB-116E, the ability to adjust the height allows easy transportation and care of the baby, such as changing diapers.

Specification

	KB-115E	KB-116E (with Hi-lo function)
Dimensions	L862mm x W525mm x H920mm	L862mm x W525mm x H810-1060mm
Tilt Angle	0° to 12°	0° to 12°
Color	Green	Green



PB-1100 / 1200

The PB-1100 / 1200 bassinet carts are also available.

Specification

	PB-1100	PB-1200
Dimensions	L755mm x W438mm x H868mm	L755mm x W438mm x H868mm
Tilt Angle	15°	15°
Color	Pink	Blue

*Please contact Paramount Bed or our distributors for more information on our bassinet carts.