


NOW ONE MASK

Delivers 24% to 90% Oxygen !



A PARADIGM SHIFT IN OXYGEN THERAPY

ONE MASK DELIVERS 24% TO 90% OXYGEN

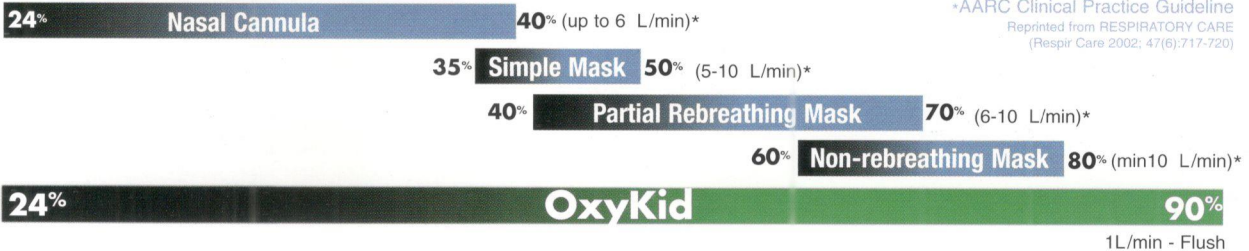

*AARC Clinical Practice Guideline
Reprinted from RESPIRATORY CARE
(Respir Care 2002; 47(6):717-720)

"From Admission to Discharge"
IDEAL FOR EMS, HOSPITAL AND HOMECARE

OxyKid's logical advantages:


- The only O₂ therapy device to deliver FiO₂ of 24% - 90%
- Can be utilized to deliver High Flow O₂ therapy to flush
- Zero probability of creating intrinsic PEEP on flow rates > 15 LPM
- Zero probability of CO₂ rebreathing on low flow rates
- Allows open communication with your patient
- Designed for nose and mouth breathers
- Convenient swivel elbow to reposition tubing
- Nasal Gastric tube can be threaded through OxyMask
- Allows for suctioning & oral care through mask
- Open mask allows for delivery of other respiratory therapies
- Reduces probability of aspiration

Cost saving advantages:

- Reduces number of O₂ devices per patient
- Simplifies and reduces inventory
- Reduces PVC waste
- Reduces manpower due to simple FiO₂ / flow changes
- Achieve higher FiO₂ on lower flow rates
- Reduces medical gas consumption


Proven to Deliver Effective Heliox Therapy

OxyKid O₂ % Guidelines

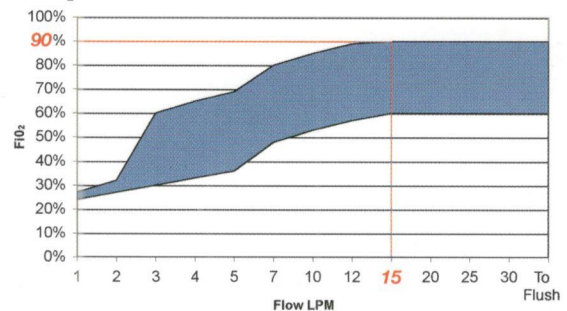

Actual FiO₂ will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.
Actual flow rate required should be prescribed in conjunction with the use of a pulse oximeter.

OxyKid™
An OxyArm™ Product

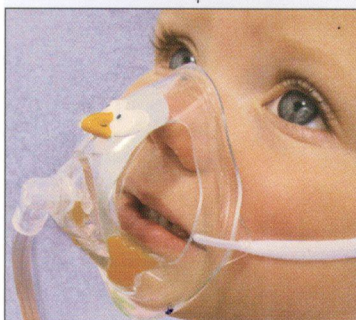

Product # OT-1025-8

Ask about OxyTyke Mask

- Allows open communication with your patient
- Designed for nose and mouth breathers
- Convenient swivel elbow to reposition tubing
- Nasal Gastric tube can be threaded through OxyTyke
- Allows for suctioning & oral care through mask

OxyTyke™
An OxyArm™ Product

Oxygen Flow Settings	Approximate O ₂ Concentrations
.25 L/min	22%
.50 L/min	25%
.75 L/min	30%
1 L/min	35%
1.5 L/min	43%
2 L/min	47%
3 L/min	50%

Southmedic
1-800-463-7146 Toll Free in North America
1-705-726-9383 1-705-728-9537 Fax
email: smedic@southmedic.com
www.southmedic.com www.oxyarm.com

50 Alliance Blvd., Barrie, ON., Canada L4M 5K3
ISO 13485

1st Open Oxygen System™
We are the experts!
Patented

OXYKID IS A REGISTERED TRADEMARK



10-2/09